

OSTEOBOOST[®]

R E S O R B A B L E B E A D K I T (R B K)

COMPLEX PROBLEM
SIMPLE SOLUTION



OSTEOREMEDIES[®]
ADVANCED MEDICAL TECHNOLOGIES

- 100% synthetic resorbable bone void filler material:
 - Calcium sulfate
 - Tricalcium phosphate
 - Hydroxyapatite
- Biocompatible
- Neutral pH
- Proprietary triphasic bone void filler
- Convenience kit for easy-to-prepare beads

Learn More At OSTEOREMEDIES.COM • 800-OSTEO-XL • 901-453-3141

OSTEOBOOST® RESORBABLE BEAD KIT

Complex Problem - Simple Solution

OSTEOBOOST® RBK utilizes a time-tested bone void filler with over 10 years clinical use to fill voids and gaps not intrinsic to the stability of the bone structure.

PREPARATION TECHNIQUE



STEP 1

Pour all of the distilled water into the powder.




STEP 2

Mix thoroughly within 1 minute.



STEP 3

Transfer the material into the mold for shaping into beads.

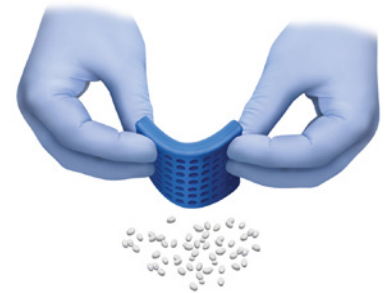

STEP 4

Let putty harden for at least 5 minutes and gently bend the mold to release beads.




Sterile Resorbable Bead Convenience Kit


- Bone Void Filler Powder
- Sterile Water
- Mixing Spatula
- Bead Mold

OSTEOBOOST® RBK's unique elongated bead shape is available in two kit sizes and three bead sizes.

5cc Kit with small and medium bead mold

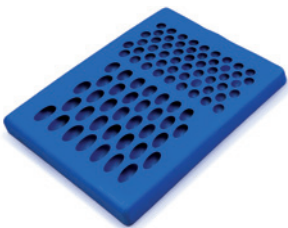

10cc Kit with medium and large bead molds

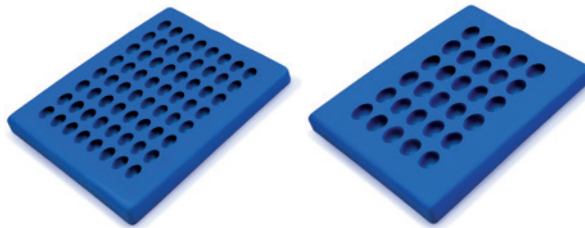

OsteoRemedies, LLC | 6800 Poplar Avenue | Suite 120 | Memphis, TN 38138

1-800-OSTEO-XL | 901-453-3141 | info@OsteoRemedies.com | OsteoRemedies.com

OsteoRemedies, OsteoBoost and the corporate mark are registered trademarks of OsteoRemedies, LLC
Support materials on file. ©2017 • 5-17-V2



OSTEOREMEDIES®
ADVANCED MEDICAL TECHNOLOGIES