

R E S O R B A B L E B E A D K I T (R B K)

OSTEOBOOST[®]

SELECT[™]

ONE MAT WITH THREE BEAD SIZES



MEDIUM
(5.5 x 9.5mm)



SMALL
(4.5 x 7mm)



X-SMALL
(3 x 4.5mm)



OSTEOREMEDIES[®]
ADVANCED MEDICAL TECHNOLOGIES

- 100% synthetic resorbable bone void filler material:
 - Calcium sulfate
 - Tricalcium phosphate
 - Hydroxyapatite
- Biocompatible
- Neutral pH
- Proprietary triphasic bone void filler
- Convenience kit for easy-to-prepare beads

OSTEOBOOST® SELECT™ RESORBABLE BEAD KIT

Complex Problem - Simple Solution

OSTEOBOOST® SELECT™ RBK utilizes a time-tested bone void filler with over 10 years clinical use to fill voids and gaps not intrinsic to the stability of the bone structure.



Sterile Resorbable Bead Convenience Kit

- Bone Void Filler Powder
- Distilled Water
- Mixing Spatula
- Spreading Spatula
- Bead Mat

OSTEOBOOST® SELECT™ RBK's unique elongated bead shape is available in three bead sizes and one double-sided bead mat for both 5cc and 10cc kits.

Front of Mat with Small and Medium Beads

Small
(4.5 x 7mm)

Medium
(5.5 x 9.5mm)

Back of Mat with X-Small Beads

X-Small
(3 x 4.5mm)

PREPARATION TECHNIQUE



STEP 1

Pour all of the distilled water into the powder.




STEP 2

Mix thoroughly for 30 seconds.



STEP 3

Transfer the material into the mat for shaping into beads.

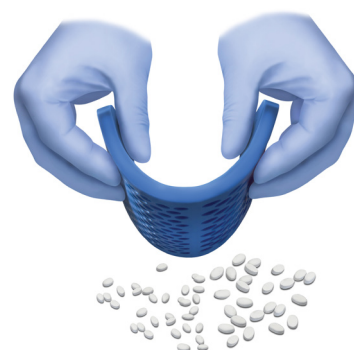

STEP 4

Let putty harden for at least 5 minutes and gently bend the mat to release beads.

OsteoRemedies, LLC | 6800 Poplar Avenue | Suite 120 | Memphis, TN 38138

1-800-OSTEO-XL | 901-453-3141 | info@OsteoRemedies.com | OsteoRemedies.com

OsteoRemedies, OsteoBoost and the corporate mark are registered trademarks of OsteoRemedies, LLC
Support materials on file. ©2020 • MKT032 • 7-20-V1

